

all-in-ONE™ Enzymatic Enzyme Detergent Surgical Instrument Cleaner with Conditioners

Highly concentrated, fast, and cost effective presoak and cleaner.

The all-in-ONE™ is Neutral pH, non-irritating to eyes and skin, and is 100% biodegradable.

Image: replacing multiple products, lowering costs, improving the usable life and performance of surgical instruments, while you improve turnaround. The all-in-ONE™ rapidly breaks down and removes all forms of proteinaceous bioburden, cleans and conditions the surface, enhances the “passive layer” protection of surgical stainless steel, lubricates moving parts, and then rinses clean. One gallon of the highly concentrated all-in-ONE™ delivers 512 gallons (1938 liters) of Cleaning with Conditioning power. The Dilution rate is .25 to .5 ounces per gallon (1.9 ml liter to 3.8 ml per liter) depending on bioburden level, encrustation, and STAINS.

The all-in-ONE™ easy FOAM-it™ lays down a thick blanket of long lasting foam to hydrate and remove all forms of proteinaceous debris, hard water deposits, and encrusted STAINS, for fast cleaning action. When water is added to the easy FOAM-it™ the ‘FOAM’ dissipates providing a clear ‘free-rinsing’ cleaning and conditioning solution.



easy FOAM-it™
Surgical Instrument
Cleaner and Conditioners

ONE application is all that is needed for thorough Cleaning and Conditioning. Apply it ONCE. It's all you really need for removing STAINS and dried on encrustation. Replace Cleaning and Conditioning products. The all-in-ONE™ is designed for use with: soaking, pre-cleaning, manual cleaning, ultrasonic cleaners, scope cleaners, and automated surgical instrument washers.



all-in-ONE™
Surgical Instrument
Cleaner with Conditioners

The all-in-ONE™ Surgical Instrument Cleaners deliver a unique formulation of detergents and multi-tiered "high-level" enzymes (Protease, Amylase, Lipase, Carbohydrase) within a Neutral pH surfactant chemical complex. The all-in-ONE™ cleaners will: break down bioburden, clean the surface, remove STAINS, enhance the "passive layer" protection of surgical stainless steel, and lubricate moving parts. The highly concentrated all-in-ONE™ cleaners are aggressive against your most difficult cleaning challenge and guaranteed to be safe for all types of metal, rigid and flexible scopes, lenses, microsurgical instrumentation and reprocessing equipment.

UPS-FS-32 easy FOAM-it™ 32 ounce FOAM applicators: 12 per case
Ergo UPS1-1 all-in-ONE™ : 4 gallons per case
Ergo UPS1-5 all-in-ONE™ : 1 ea. 5 gallon jug
Ergo UPS1-15 all-in-ONE™ : 1 ea. 15 gallon drum
Ergo UPS1-30 all-in-ONE™ : 1 ea. 30 gallon drum
Ergo UPS1-55 all-in-ONE™ : 1 ea. 55 gallon drum

Dilution Rate: .25 ounce to .5 ounce per gallon depending on the level of bio-burden and STAINS.



Customer Service: 800-428-6988, Fax: 856-234-7918 email: CustomerService@ergo-Logistics.com
Product Information and Technical Support: 509-747-5027 email: Support@ergo-Logistics.com
Website: www.ergo-Logistics.com